

SimplyGo - reliability and maintenance made simple!

COPD (chronic obstructive pulmonary disease) patients are dependent on oxygen therapy equipment that they need for many hours* a day. **SimplyGo** offers the independence and freedom patients want in a durable design that can resist the bumps and bruises of daily activities while providing an effective therapy.



Built-in reliability

SimplyGo strives to set a new standard in portable concentrators' reliability.

SimplyGo has been designed, built, and rigorously tested to withstand the stress of everyday life when following patients anytime, anywhere.

During development, **SimplyGo** was subjected to extreme conditions including impact, vibrations, and temperature extremes to test reliable performance day-in and day-out.

With 15 000 hours of compressor lifetime, its performance is trustworthy.

This may mean fewer callbacks for you and fewer frustrations for your patients due to accidental damage.

You can count on **SimplyGo**.

Maintenance made simple!

SimplyGo makes it easy for providers, as it requires virtually no preventive or patient maintenance.

SimplyGo is flexible:

- Make a choice between two-, three-, or five-year warranty. Pick the plan that's best for your service.
- **SimplyGo**'s easily accessible parts and comprehensive service tools allow your technicians to service the device in-house instead of returning it to the manufacturer.

Notes

*Clinical guidelines recommend that COPD patients should use oxygen therapy for at least 15 hours per day to have positive effects on their long term survival and their quality of life. Therefore these patients require oxygen at home (stationary oxygen therapy) and outside (ambulatory oxygen therapy).

Contact us

Philips Healthcare is part of Royal Philips Electronics

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Philips Respironics International Headquarters
+33 1 47 28 30 82

Philips Respironics Deutschland
+49 8152 93 06 0

Philips Respironics France
+33 2 51 89 36 00

Philips Respironics Italy
+39 039 203 1

Philips Respironics United Kingdom
+44 800 1300 845

Philips Respironics Spain
+34 915 669 000

Philips Respironics Sweden
+46 8 120 45 900

Philips Respironics Asia Pacific
+852 3194 2280

Philips Respironics Australia
+61 2 9666 4444
+61 1300 766 488

Europe, Middle East, Africa
+49 7031 463 2254

Respironics, **SimplyGo**, and EverFlow are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/



© 2012 Koninklijke Philips Electronics N.V. All rights reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication. CAUTION: US federal law restricts these devices to sale by or on the order of a physician. Printed in the United States - APRIL 2012. PN 1098670 MCI 4104723



Life made simple

Philips Respironics **SimplyGo** portable oxygen concentrator - simply changes your patient's life

PHILIPS
RESPIRONICS
sense and simplicity

SimplyGo, the freedom your patients deserve!

A short shopping spree, a museum visit, a pleasant weekend, or a well deserved holiday, with SimplyGo your oxygen-therapy patients may enjoy it all. Lightweight, and durable, delivered with a handy carrying case and a cart with easy-to-go wheels, SimplyGo is always ready for a stroll. Its accessory case keeps everything on hand, for all occasions.



SimplyGo is Philips first oxygen concentrator that offers the benefits of a continuous flow mode in a lightweight, easy-to-carry design of 4.5 kg. No need to compromise between an effective oxygen therapy output for your patients and the usual weight of the device. SimplyGo offers a solution that fulfills the needs of most of your oxygen-therapy patients. With SimplyGo, patients have the flexibility to be away for the day, for work, for a weekend or a long holiday.

Three oxygen-delivery modes are available to fit most patients needs:

- Pulse mode** SimplyGo detects breath intake and delivers the necessary volume of oxygen.
- Continuous flow** Continuous oxygen delivery from 0.5 to 2LPM.
- Sleep mode** Pulse mode has an increased trigger sensitivity for use during sleep. SimplyGo switches automatically to 2LPM *Continuous flow* if no breath intake is detected.

With oxygen always on hand, your patients have the freedom to go anywhere, anytime.

At home, the **EverFlo** stationary concentrator is the ideal solution to complement the **SimplyGo** ambulatory device for patients requiring home oxygen therapy. Since **SimplyGo** and **EverFlo** use **non-delivery technology**, they eliminate the inconvenience of recurrent oxygen deliveries and can help improve patients' independence and quality of life.*

*Portable oxygen increases the daily duration of oxygen therapy in patients with COPD and improves the quality of life of the most compliant patients. Vergeret et al. Eur Respir J 1989;2: 20-25



SimplyGo at-a-glance



SimplyGo, ready for a stroll!



SimplyGo specifications

Product specifications	
Oxygen concentration ¹	87 to 96% at all settings
Flow settings	
Pulse-mode bolus size	1 = 12 ml 4 = 48 ml 2 = 24 ml 5 = 60 ml 3 = 36 ml 6 = 72 ml
Continuous-flow mode	0.5 LPM - 1 LPM - 1.5 LPM - 2 LPM
Night mode	Variable pulse volumes based on breath rate, to maintain a constant minute volume per setting: 1 = 250 ml/min 2 = 500 ml/min 3 = 750 ml/min 4 = 1 000 ml/min 5 = 1 250 ml/min 6 = 1 500 ml/min
Sound level	43dB (nominal) at pulse setting 2
Alarms	Low oxygen purity (<83%) No breath High breath rate No flow Low battery Depleted battery System technical fault
Dimensions	29.2 cm x 25.4 cm x 15.2 cm 11.5 in x 10 in x 6 in
Weight	4.5 kg 10 lbs. with battery installed
Power consumption	150W while charging 120W while not charging
Operating conditions	
Operational temperature	5° to 40°C 41° to 104°F
Relative humidity	15 to 95%
Altitude	Up to 3 048 m 10 000 ft
Battery autonomy	
Pulse mode setting ²	1 2 3 4 5 6 Continuous flow setting 0.5 1 2
1 battery	3 h 42 3 h 30 3 h 06 2 h 42 2 h 18 1 h 54 3 h 06 2 h 18 0 h 42
2 batteries	7 h 24 7 h 00 6 h 12 5 h 24 4 h 24 3 h 18 6 h 12 4 h 36 1 h 24
3 batteries	11 h 06 10 h 30 9 h 18 8 h 06 6 h 54 5 h 42 9 h 18 6 h 54 2 h 06
4 batteries	14 h 48 14 h 00 12 h 24 10 h 48 9 h 12 7 h 36 12 h 24 9 h 12 2 h 48
Charging time	2 - 3 hours / battery from fully discharged

¹Based on an atmospheric pressure of 14.7 psia (101 kPa) at 21°C | 70°F
²Based on 20 breaths per minute

Ordering information

SimplyGo international system	Part Number
Includes: SimplyGo portable oxygen concentrator, carrying case, mobile cart, one rechargeable Lithium Ion battery, DC and AC power cords, accessory case and user manual	1069058
Accessories	Part Number
Carry cart	1074885
SimplyGo carrying case	1082663
Lithium Ion battery	1082662
Power cord (Standard Europe)	1082665
External battery charger (European plug)	900-118
External battery charger (UK/Ireland plug)	900-119
Accessory case	1083696
AC supply	1082661
DC car charger	1083692
DC airline cord	1083693
Humidifier pouch	1083699
Humidifier connecting tube	1039642
Warranty options	Part Number
SimplyGo two-year total warranty	1097623
SimplyGo three-year total warranty	1097577
SimplyGo five-year total warranty	1097778